


ON-Q[®] is Redefining Recovery™

It's as easy as these 5 checks...

1. Keep CLAMP(S) OPEN, unless directed otherwise by surgeon
2. Tubing NOT KINKED
3. Filter NOT TAPED OR COVERED
4. Flow restrictor in direct contact with or TAPED to skin, away from cold therapy
5. If removing catheter, pull gently close to the skin and CHECK THE DISTAL END OF THE CATHETER FOR BLACK MARKING

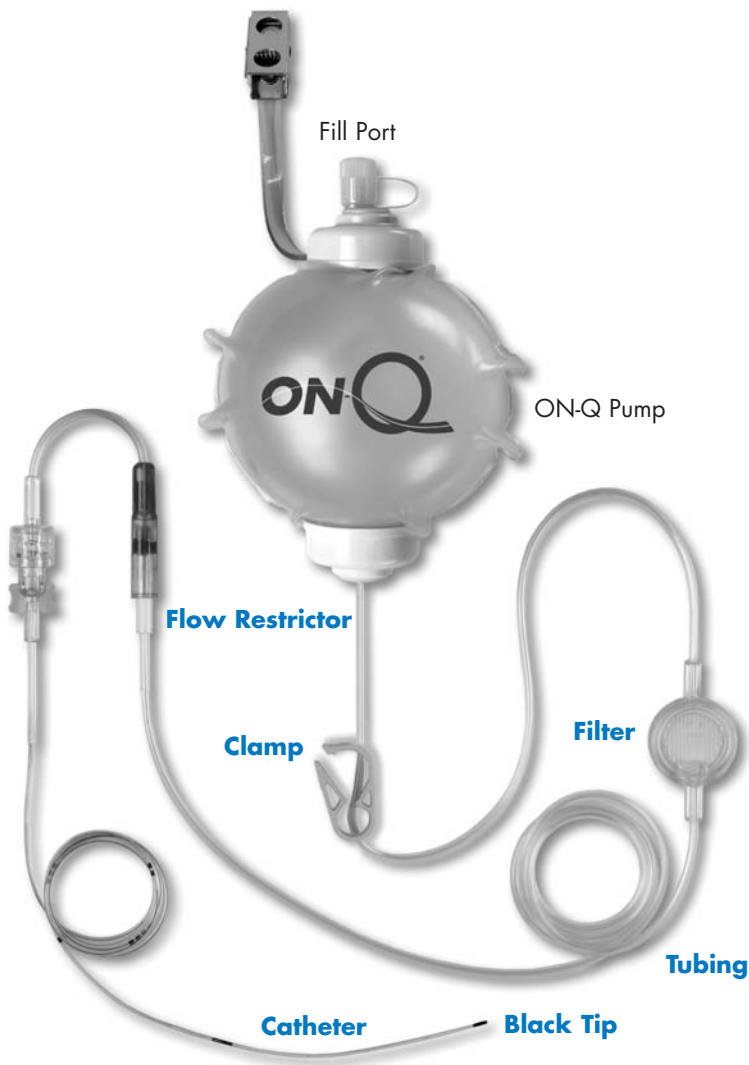

For additional information,
call our 24-hour product support hotline:
800-444-2728
or call your local Sales Representative at:

This inservice does not replace the product Directions for Use (DFU). It is imperative that the clinician read and understand the product DFU. The DFU contains detailed product information including important cautions, warnings and contradictions.

ON-Q[®] is

Redefining Recovery[™]

Now with expanded indications based on clinical outcomes:

- significantly better pain relief than narcotics alone
- significantly less need for narcotics post surgery

What you may expect clinically:

- Quicker return to normal, including ambulation, body functions, and full range of motion
- Decreased complications associated with narcotics
- Clinically proven success—used in hundreds of thousands of surgeries
- Over 90% patient satisfaction

What you may expect day to day:

• How can I tell that the pump is infusing?

Infusion may not be evident for more than 24 hours after surgery. Over time, the outside bag will loosen and begin to crease. As the medication is delivered, the pump balloon will *gradually* become smaller.

• When is the infusion complete?

Infusion is complete when the delivery time has passed and the pump is no longer inflated. A hard tube will be present in the middle of the pump. The infusion will typically last for 2 to 5 days, depending on the size and volume of the pump.

• Can the pump be refilled?

No. The pump is for single use and is disposable.



www.AskYourSurgeon.com



Rx Only

20202 Windrow Drive, Lake Forest, CA 92630

Phone: 800-448-3569/949-206-2700

For a list of supporting studies, please visit www.iflo.com.

I-Flow, ON-Q, PainBuster, and www.AskYourSurgeon.com are registered trademarks of I-Flow Corporation.

Redefining Recovery is a trademark of I-Flow Corporation.

© 2005 I-Flow Corporation. All rights reserved.

11/2006 1303851B