


## Compatibility of Life-Tech ProLong™ and ContiStim™ Continuous Nerve Block Needles with Non Life-Tech Stimulators

Life-Tech Stimulators, model EZStim (ES-400) and Tracer III (NL-3) connect to stimulating needles and catheters by way of a 1.5 mm conducting lead wire. This is compatible with Life-Tech stimulating needles. However, competitive stimulators (B.Braun and Pajunk) have a 2 mm lead wire connection to accommodate their respective stimulating needles and catheters.


The following is guidelines for use of Life-Tech needles with other brand stimulators.

**B. Braun Stimplex HNS11** and **Pajunk MultiStim Vario** requires the use of the Life-Tech RBW-5L cable to connect the needle or catheter. This is the same cable that connects to the Life-Tech EZ Stim and Tracer III Stimulators.

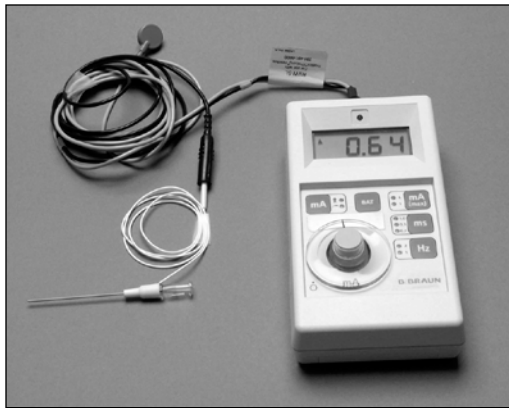

The B. Braun Stimplex HNS11 connected to the RBW-5L cable and ProLong stimulating needle.

**B. Braun Stimplex Dig RC** has a cable that is permanently connected to the unit. Therefore, instead of using the RBW-5L cable, a BA-01 adaptor can be used at the terminal end of the cable to adapt to a Life-Tech needle or catheter.

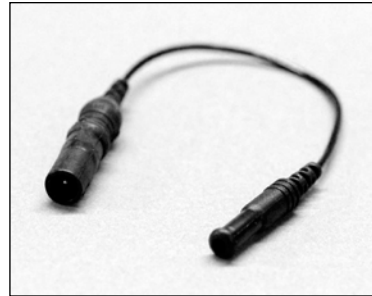

BA-01 Adaptor

In summary:

If the needle is	Connected to this stimulator	Use this cable/ adaptor
ProLong PL(T)-xxxC or ContiStim CS(T)-xxx	B. Braun HNS-11	RBW-5L
ProLong PL(T)-xxxC or ContiStim CS(T)-xxx	Pajunk MultiStim Vario	RBW-5L
ProLong PL(T)-xxxC or ContiStim CS(T)-xxx	B. Braun Dig RC	BA-01

Please contact the Clinical Services Department at 800-448-3569 or 949-206-2700 if you have any questions regarding this information.